



Hole layout table

TAG	X LOC	Y LOC	SIZE	See instalation table
C1	-86,50	70,50	65 X 100	B, C & H
C2	38,50	-95	65 X 100	D, E, F & G
H1	-275	-88	Thread with insert	A
H2	-275	88	Thread with insert	
H3	-236	-160,50	Thread with insert	
H4	-236	160,50	Thread with insert	
H5	-54	-160,50	Thread with insert	
H6	-54	160,50	Thread with insert	
H7	136	-160,50	Thread with insert	
H8	136	160,50	Thread with insert	
H9	178	-88	Thread with insert	
H10	178	88	Thread with insert	

Instalation table

TAG	Instalation
A	Monteringshuller i gulvet - 10 mm. Trous de montage - 10 mm. Montierungslöcher - 10 mm. Mounting holes - 10 mm.
B	Koldt vand - R 3/8" Amenée d'eau - R 3/8" Wasserzuluß - R 3/8" Water inflow - R 3/8"
C	Trykluft - R 3/8" Air comprimé - R 3/8" Preßluft - R 3/8" Air inflow - R 3/8"
D	Net 230 volt - 3x1.5 Secteur 230 volt - 3x1.5 Netz 230 volt - 3x1.5 Power supply 230 volt - 3x1.5
E	Klokkekald - 2x1.5 Ligne de sonnerie - 2x1.5 Klingelleitung - 8x0.5 Bell lead - 2x1.5
F	Styreledning til sug - 2x1.5 FConducteur pour pompe aspirante - 2x1.5 Schaltleitung für Saugmaschine - 5x1.5 Control lead for suction machine - 2x1.5
G	externe tilslutninger: skærm, netværk & USB connexions externes: écran, réseau et USB externe Verbindungen: Blitzschirm, Netzwerk und USB external connections: screen, network & USB
H	Sug - 32 mm Conduit d'aspiration - 32 mm Saugleitung - DN40 Suction pipe - 32 mm